

autonomEgo.®

QUICK START GUIDE



ACCESSIBILITY
SERVICES, INC.

800.933.8400

accessibilityservices.com

autonomEgo[®]

QUICK START GUIDE

This document provides step-by-step instructions to staff on how to properly arrange, connect, & start up the *autonomEgo* system.

GET STARTED

1 Roll the *autonomEgo* unit to the user's preferred side of the bed & lock the casters in place.




2 Connect the 1/4" Nurse Call Cable from the back plate of the unit to the wall mounted alert.



3 Connect the phone cord from the back plate to an analog phone jack.



4 Connect the DB25 cable to the back plate for bed control. Plug the other cable into the bed's hand pendant port.

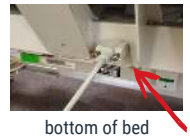

bottom of bed



ACCESSIBILITY
SERVICES, INC.

SET UP CONTINUED

- 5** Remove Sip & Puff (Dual) from the side holder on the *autonoMEgo* unit & attach it to the bed side rails.




- 6** Connect the cords from the Sip & Puff (Dual) into the corresponding jack on the back plate of the unit. *The cables are labeled "S/P" & "MICROPHONE".




- *** For SWITCH use, unplug the Sip & Puff (Dual). Plug a 3.5mm splitter into the jack on the back plate. This will allow the use of individual switches. Plug the switches into the splitter.



GET IN TOUCH

 @accessibilityservices

 @accessibilityservices

CONTACT US AT 800 933 8400
OR VISIT [ACCESSIBILITYSERVICES.COM](https://www.accessibilityservices.com)

SET UP CONTINUED

7 Plug the power cord at the base of the *autonoMEgo* unit into a nearby grounded outlet.



8 Turn on the *autonoME* ECU box by switching the red button from "0" to "1".



9 Once the computer starts up, the *autonoME* software will open automatically. If it does not start, you can manually open it by selecting the icon in the taskbar, near the **Start** button.





10 On the **Main Menu** of the *autonoME* Software choose "**Settings**" & set-up the user's preferred input method(s), i.e., **voice, switch/sip & puff, &/or eye gaze.**




GET IN TOUCH

CONTACT US AT 800 933 8400
OR VISIT ACCESSIBILITYSERVICES.COM


 @accessibilityservices
 @accessibilityservices

SET UP COMPLETE



11 Position the screen at the appropriate distance & height for the user. *When using eye gaze, the optimal distance is **between 23" & 26"**.

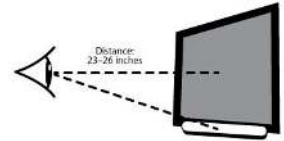

12 Turn the IR device (UIRT) located at the top of the *autonoMEgo* unit & position towards the room television in order to control the TV with the device.

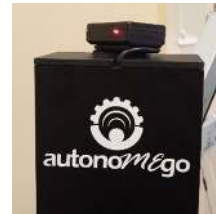

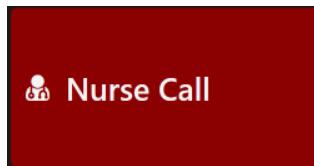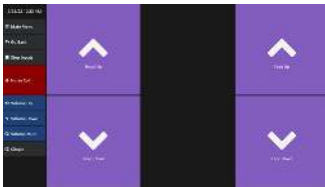

13 Have the user test the **bed controls, nurse call, & other functions.**



GET IN TOUCH

CONTACT US AT 800 933 8400
OR VISIT ACCESSIBILITYSERVICES.COM

 @accessibilityservices
 @accessibilityservices

REFERENCE IMAGES

1



1/4" Nurse Call Cable

2



Analog Phone Cable

3



DB25 Cable

4



Sip & Puff (Dual) Cords



ACCESSIBILITY
SERVICES, INC.

autonomEgo[®]

QUICK START GUIDE

is complete

**For further assistance with your autonomEgo
system call or connect**

800.933.8400

accessibilityservices.com